Vacsure

SAFETY MULTI-SAMPLE NEEDLE

ULTRA SHARP NEEDLE


 Our VacSure Needle is a medical high activity grade silicone, stainless steel needle with a dimond cut beveled edge. Our needle is harmless to human skin & perfect for blood collection. Our silicone coating provide lubricity and a protective coating for the skin in dermatological treatments and pharmaceutical applications.



PERFECTLY ALIGNED WITH BEVELED EDGE OF THE NEEDLE

The **VacSure Safety Multi-Sample Needle** provides streamlined and effective injection safety.

- •Activation that requires virtually no change in clinical technique for easy integration.
- •**Designed** to specifically reduce the risk of accidental needlestick injuries.
- •Allows for shielding of the needle tip immendiately after injection, with single finger stroke or can be use on a flat surface for safety activation.
- •Audible click confirm activation to support your safety.

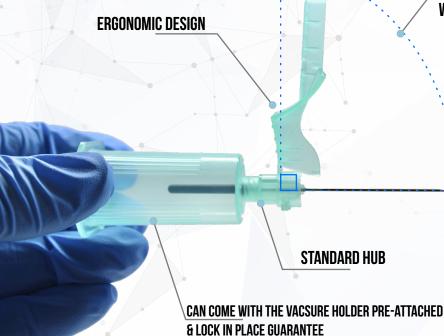



Screw VacSure Safety Multi-Sample Needle into any needle & tube holder until it's completly secure



Put the VacSure Safety Shield of the needle on a flat surface.

While holding the needle holder, apply gently pressure to the shield until a click sound is heard.

The VacSure Safety Multi-Sample Needle is completly

secure and ready to be disposed on a Sharp Container.



Put the thumb on the **Edge of VacSure Safety Shield** of the needle. While holding the needle holder with the other fingers, apply gently pressure to the shield until a click sound is heard.

The **VacSure Safety Multi-Sample Needle** is completly secure and ready to be disposed on a Sharp Container.

	Poduct Code	Item Description	Needle Gauge	Needle Length	Shield Color	Quantity
\	VM-1021 VM-1021-AH (w. Holder)	Safety Multi-Sample Needle	21 G] 1/4"		Box: 48 Case: 2,400 Box: 100 Case: 500
	VM-1022 VM-1022-AH (w. Holder)	Safety Multi-Sample Needle	22 G] 1/4"	•	Box: 48 Case: 2,400 Box: 100 Case: 500









